

Information for the attending physician

Marking of the fracture site for treatment with Melmak ultrasound

- Determination of the fracture site/treatment site by marking (with a pen on the skin) under an X-ray C-arm or measuring on digital X-ray images.
- Plates, screws, nails, intramedullary nails or similar used to stabilize the fracture are not a contraindication.
- Metal plates reflect the ultrasound. In this case, treatment must be carried out from the side or opposite.
- No heating, since it is pulsed ultrasound; therefore also suitable for infected fractures.
- Several treatment sites are possible, number of treatments with the therapy device is not limited.
- Sonication only in the transducer area, no lateral scattering.
- 20 minutes daily application per treatment site over the fracture gap without moving the transducer.



If you have any questions, please do not hesitate to contact the Melmak medical product consultants in the office. We look forward to your call.

Melmak GmbH
ultrasound for fracture healing

Ursula Fischer

ppa. Ursula Fischer

Do not treat with Melmak Ultrasound or only after prior consultation with your doctor, in case of

- Cancer
- Children/adolescents whose skeleton is not yet fully grown (epiphyseal plate)
- Fractures of the skull or spine
- Pacemakers or similar
- Pregnancy

**Wir sind IHK
Ausbildungsbetrieb**

MD: Florian Hagenmeyer • Registry Court: München HRB 183870
VR-Bank Landsberg-Ammersee eG • IBAN: DE52 7009 1600 0000 0250 38 • BIC: GENODEF1DSS

